


# CPSI Integration Guide

## CR1100 & CR1500

- Not Compatible with Other Models -

VERSION 02 – RELEASE DATE JUNE 2019

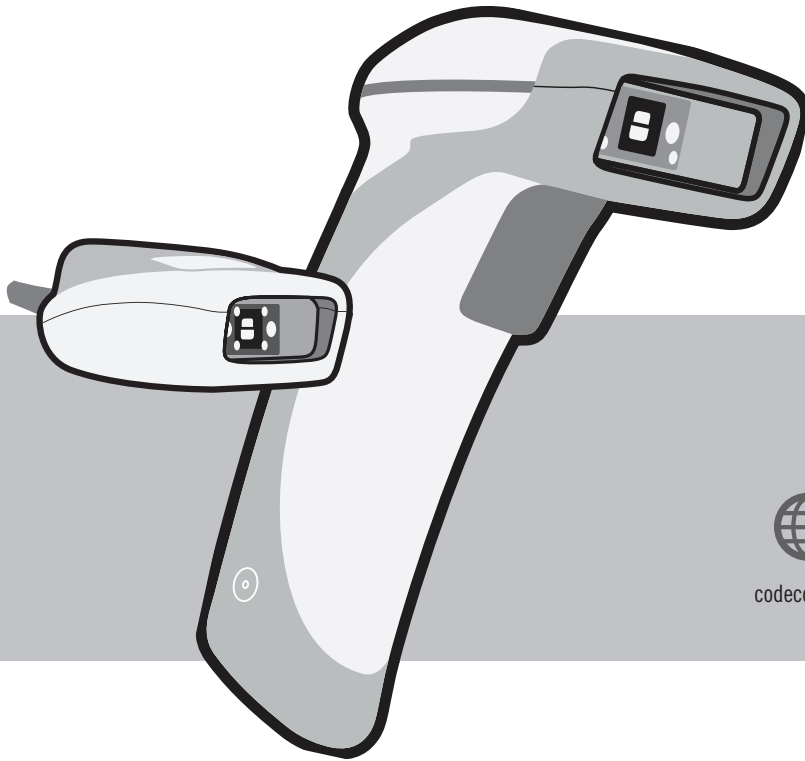

[codecorp.com](http://codecorp.com)



Configuration Guide



[codecorporation](https://www.youtube.com/channel/UC0e0111111111111111111111)

## Configuring CR1100 and CR1500 Barcode Readers

To configure a CR1100 or CR1500 to be dedicated to a patient room or workstation on wheels, scan the following barcode:



M20219\_02

## CR1500 Feedback Settings

Scan the preferred feedback settings (these will not work on a CR1100):



M20329\_01

Beep On  
Vibrate On



M20331\_01

Beep On  
Vibrate Off



M20330\_01

Beep Off  
Vibrate On

---

## CR100 and CR1500 Reader Settings



M20111\_01

Reset to factory defaults

## Firmware Version Information

Scan the following barcodes to identify firmware versions:



M20361\_01

Reader information